Usage Guide

Congratulations! You are the proud owner of a menstrual cup! They will faithfully accompany you during the period...

Before you use the menstrual cup for the first time please boil them. To do this, bring a pot or water heater to the boil and then throw the AvaLoona Cup into it. Don't worry, the silicon will not deform.

It is recommended to practice the correct application BEFORE your period starts. So you can try it without any stress and hassle and get a feel for it.

Before inserting the AvaLoona Cup, you have to "fold" it, so that it is hardly bigger than a tampon. Hold it in one hand. The insertion. This works both sitting, as well as standing or crouching. Just try to find which feels most comfortable. With your free hand, open the labia. Then insert the AvaLoona Cup, so far that the stem is not peeping out. Do not insert it further than 1 cm from the vaginal opening, because that may cause leaks. And removing it will be harder.

The AvaLoona Cup should have now opened. You can try turning it a little. If that goes without any problems, then it should have opened correctly by itself.

The whole procedure should not hurt at any point. If you are very sensitive, you should use some (water-based) lube. Normally this should not be necessary. If the stem irritates the vagina, you can slightly trim the end of the stem. Removing it should be no problem anyway. But do not cut off too much!

The removal. This should be done stress free in a comfortable position. Grip the stem and carefully pull it out. If you reach a little deeper so that you can feel the cup, then you can place a slight pressure on it to remove the vacuum. Like that the AvaLoona Cup can be removed much more easily. The contents can be emptied into the toilet. Then the AvaLoona Cup should be cleaned under running water. You can also use ph-neutral care products. Simply empty it 2-3 times per 24 hours, wash and reinsert it. After the period, boil your cup.

If you need more info, simply search for "How to use a menstrual cup" on YouTube. For questions or concerns, ask your gynecologist before using!

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